


Valiant 2 rehab XXL

Robust and reliable treadmill for rehabilitation

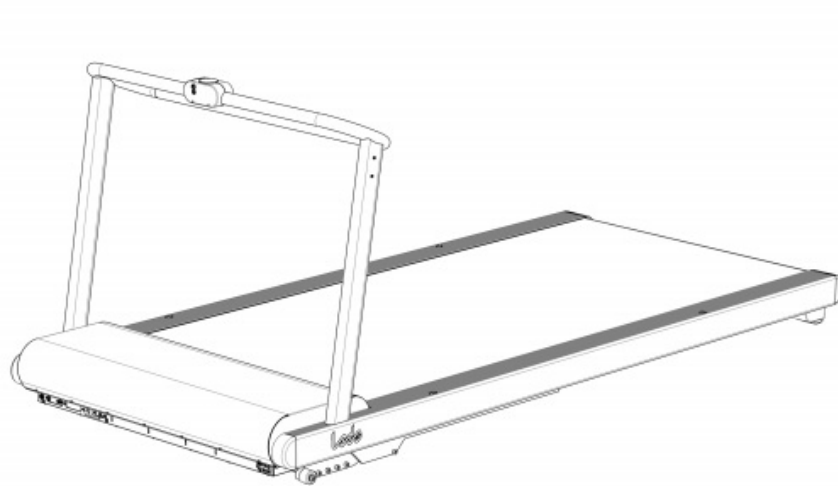

Highlights

High standards

Lode is a socially and environmentally responsible company. All Lode products are RoHS/WEE compliant and Lode is ISO 9001:2003, ISO 13485:2008 and FDA 510K certified. All medical products comply to MDD 93/42/EEC, incl. IEC 60601-1.

Various handrails available

Various handrails are optional available, making the treadmill suitable for your specific stress test setting.

Smooth acceleration

All treadmills have a smooth acceleration from 0 km/h to the start speed.

Extremely accurate

The treadmill is extremely accurate in its speed and angle settings

Patient friendly treadmill

The treadmill offers ultimate comfort for the patient:

- low step-up height
- faultless operation
- smooth acceleration
- reliable and reproducible test results
- low noise



Valiant 2 rehab XXL



Robust and reliable treadmill for rehabilitation

Modern designed treadmill specifically designed for rehabilitation. The low step-up height makes the treadmill perfectly suitable for all test subjects in the field of rehabilitation. The emergency stop with magnetic lanyard on the standard front handrail provides additional safety for the user. The Valiant 2 rehab offers a smooth acceleration from 0 km/h and is continuously adjustable in a range of 0,1 – 20 km/h. The network module provides control within a Lode software network (LEM, LRM or LCRM). Thanks to the unique low design of the cover plate it is possible to place a mirror or camera in front of the treadmill which makes it easier to monitor the gait of the test subject. The Valiant 2 rehab XXL has a standard running surface of 250 x 120 cm and 0-15% elevation. The treadmill is supplied with a 7" control unit with touchscreen for manual control.

A USB A-B cable only for service purposes will be standard delivered with the product.

To connect LEM, LRM or LCRM you need a special interface cable that can be ordered under part number 930930.

Features



Low noise

Material choices, refined components and accurate manufacturing techniques lead to low noise.



Robust design

The product is designed to withstand continuous heavy use by subjects in most weights and sizes.



Downhill walking as an option (-10%)

This treadmill can be executed with 10% negative elevation. This allows for downhill walking which is extremely useful for rehabilitation of certain injuries.



Optional integrated BPM and SpO2

The treadmill can be extended with a stable and reliable blood pressure module and SpO2 measurement.



LCRM compatible

This treadmill can be connected to the Lode Cardiac Rehab manager offering a complete cardiac rehab setting

17
cm

Extreme low step-up height

To allow people to safely and comfortably step up the treadmill it is important to have a very low step-up height

0.1
km/h

Small adjustment steps

The speed of the treadmill can be adjusted in the smallest steps of only 0.1 km/h!



Low cover plate

The motor compartment of the treadmill is designed in such a way that the cover is only marginally higher than the belt surface. This allows for low camera positions in case the treadmill is used for gait analysis purposes. Also a therapist has the best possible view on feet and lower extremities.



FOR LIFE | UNDERSTANDING
MOVEMENT & PERFORMANCE

Valiant 2 rehab XXL

Robust and reliable treadmill for rehabilitation



Interconnectivity between Lode products

Connecting Lode products has never been easier! Lode rehab and sports products have a standard Network card:

- To be able to connect the first product to the PC with L(C)RM a Lode proprietary network to PC cable is needed (#930930).
- From the second product onward products can be connected to the previous one, creating a bus network configuration;
- The last product always needs a termination plug to avoid interference and loss of data. Therefore all products with such a network card come with a termination plug.

Benefits




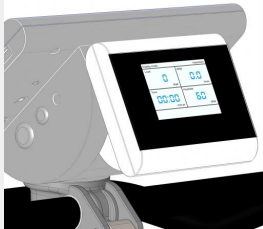











- Lossless data connection
- High bandwidth
- No interference of COM ports
- Daisy chain connection
- Full access of all data in the product to LCRM

Valiant 2 rehab XXL

Robust and reliable treadmill for rehabilitation



Valiant 2 rehab XXL can a.o be extended with the following options:
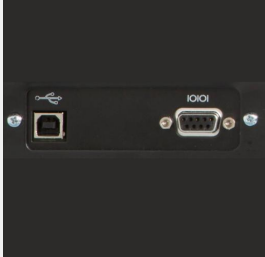
| | | | | |
|--|---|---|--|--|
| <p>Remote Control for treadmill</p> <p>Ease of use</p>  <p>Partnumber: 945831</p> | <p>Add program function to 7" touch screen for Treadmill</p> <p>Programmable</p>  <p>Partnumber: U945815</p> | <p>Table foot for Control Unit with Touchscreen</p> <p>Control from any desired place</p>  <p>Partnumber: 945818</p> | <p>Colour Display 3.5" - 2nd screen</p> <p>Multifunctionality</p>  <p>Partnumber: 945819</p> | <p>SpO2 for control unit with touch panel (treadmill)</p> <p>Oxygen saturation</p>  <p>Partnumber: 945822</p> |
| <p>SpO2 for control unit with touch panel - ordered afterwards</p> <p>Ordered afterwards</p>  <p>Partnumber: P945822</p> | <p>Blood Pressure Measurement with ECG trigger for treadmills</p> <p>with ECG trigger</p>  <p>Partnumber: 945824</p> | <p>Heart rate for treadmills</p> <p>Heart rate in beats per minute</p>  <p>Partnumber: 945820</p> | <p>Emergency Stop Button</p> <p>Ultimate safety</p>  <p>Partnumber: 945804</p> | <p>Ambient sensor pack</p> <p>Check environmental conditions during test</p>  <p>Partnumber: 945826</p> |
| <p>Universal treadmill Arm Support</p> <p>Comfort for both test subject and test</p>  <p>Partnumber: 945805</p> | <p>Extension for emergency lanyard</p> <p>Original Accessory</p>  <p>Partnumber: 945931</p> | <p>USB to Serial converter</p> <p>Easy connection</p>  <p>Partnumber: 226012</p> | <p>Network module connection cable</p> <p>Easy connection</p>  <p>Partnumber: 930930</p> | <p>Communication Module</p> <p>Connect to ECG and spirometers</p>  <p>Partnumber: 945850</p> |

Valiant 2 rehab XXL

Robust and reliable treadmill for rehabilitation



Specifications

Workload

| | | |
|----------------------------|----------|----------|
| Maximum speed | 20 km/h | 12.4 mph |
| Minimum operational speed | 0.1 km/h | 0.1 mph |
| Positive elevation | 15 % | |
| Elevation adjustment steps | 0.5 % | |

Comfort

| | | |
|---------------------|--------|---------|
| Allowed user weight | 225 kg | 496 lbs |
|---------------------|--------|---------|

User Interface

| | |
|--------------------------|---|
| English user interface | ✓ |
| Norwegian user interface | ✓ |
| Czech user interface | ✓ |
| Danish user interface | ✓ |
| Dutch user interface | ✓ |
| Finnish user interface | ✓ |
| French user interface | ✓ |
| German user interface | ✓ |
| Italian user interface | ✓ |
| Japanese user interface | ✓ |
| Korean user interface | ✓ |
| Polish user interface | ✓ |
| Portugese user interface | ✓ |
| Russian user interface | ✓ |
| Spanish user interface | ✓ |
| Turkish user interface | ✓ |
| Ukrainian user interface | ✓ |

Connectivity

| | |
|--------------------------|---|
| Optional USB connector | ✓ |
| Optional RS232 connector | ✓ |

Dimensions

| | | |
|------------------------|--------|------------|
| Walking surface length | 250 cm | 98.4 inch |
| Walking surface width | 120 cm | 47.2 inch |
| Step up height | 17 cm | 6.7 inch |
| Product length (cm) | 312 cm | 122.8 inch |
| Product width (cm) | 148 cm | 58.3 inch |
| Product height | 130 cm | 51.2 inch |

Order info

Partnumber: 938915

**Specifications are subject to change without notice.*

Power requirements

| | | |
|--|---------|-----------|
| Power cord length | 250 cm | 98.4 inch |
| Power cord IEC 60320 C19 with CEE 7/7 plug | ✓ | |
| Power cord NEMA | ✗ | |
| Maximum rated power input | 2500 VA | |
| 115 V AC 50/60 Hz (2 phases) | ✓ | |
| 230 V AC 50/60 Hz | ✓ | |
| Maximum motor power | 3.0 kW | |

Standards & Safety

| | |
|----------------------------|---|
| IEC 60601-1:2005 | ✓ |
| ISO 13485:2003 compliant | ✓ |
| ISO 9001:2008 compliant | ✓ |
| Standard emergency lanyard | ✓ |